

PROCESSING PRECI-BUT

1. Determine the path of insertion of the prosthesis.
2. Seat the **1902** paralleling mandrel on the **1904** male and place it into the surveyor.
3. Attach the male with sticky wax to the post-coping or to the bar.
4. Invest, burnout, and cast.
5. Cement the post-copings in the mouth and take impression.
6. Place the **1906** model analogues in the impression and pour a model in stone.
7. Press the **1907** processing female over the model analogue and make a refractory model.
8. Place the **1908** retention basket pattern over the replica of the processing female and incorporate it into the wax pattern of the cast metal frame.
9. Invest, burnout, and cast.
10. The retention basket is now cast in metal and will considerably reinforce the acrylic resin covering the attachment.
11. Finish the frame.
12. Seat the original **1903** female with space maintainer onto the model analogue and finish the prosthesis.

Side effects, warnings and precautions

- The attachments are intended for single use.
- The products are non-sterile.
- There is a risk of poor fit when patient conditions change.
- Bacterial adhesion can be avoided by applying hygiene measures.
- Inappropriate use or bad manufacturing can lead to premature wear of the attachments.
- The functionality of the attachments will be adversely affected by traumas such as grinding and bruxism.
- For the purpose of traceability we advise you to record the lot number of the applied products in the patient file.
- Do not heat items containing titanium.
- Do not use items containing nickel in case of nickel allergy.
- The accessories RE H 79 and H 35 must be used outside the mouth.