

PROCESSING PRECI-POST

1. Prepare the root canal with the standardized **2004G/2004B** yellow or blue root canal reamer and place the corresponding **2002G/2002B** yellow or blue opaque impression post in the canal.
2. Take impression. The posts are provided with retentions for a proper fit in the impression material.
3. Pour a working model.
4. Whether the impression is plated or an acrylic resin model is made, the opaque impression post does not bond with any of these materials.
5. Replace in the laboratory the impression post with the transparent **2003G/2003B** yellow or blue laboratory post. Reduce to its correct length and make sure that the top offers a little retention for wax.
6. Make the wax pattern, invest, burnout, and cast. The result is always an exactly fitting post-coping without soldering joint.

Side effects, warnings and precautions

- The attachments are intended for single use.
- The products are non-sterile.
- There is a risk of poor fit when patient conditions change.
- Bacterial adhesion can be avoided by applying hygiene measures.
- Inappropriate use or bad manufacturing can lead to premature wear of the attachments.
- The functionality of the attachments will be adversely affected by traumas such as grinding and bruxism.
- For the purpose of traceability we advise you to record the lot number of the applied products in the patient file.
- Do not heat items containing titanium.
- Do not use items containing nickel in case of nickel allergy.
- The accessories RE H 79 and H 35 must be used outside the mouth.