

REPLACING A PRECI-HORIX RIDER

PRECI-HORIX attachments are supplied with the **1802** yellow standard riders.

If the retention of the rider no longer meets the requirements, it can be replaced with the **1805** white rider with reduced retention or the **1806** red rider with increased retention.

1. Cut the riders with a scalpel in two and remove them from the prosthesis.
2. Position the rider with the desired retention on the supplied **1705** insertion tool and press it in position.
3. Check if the required retention is attained.

Side effects, warnings and precautions

- The attachments are intended for single use.
- The products are non-sterile.
- There is a risk of poor fit when patient conditions change.
- Bacterial adhesion can be avoided by applying hygiene measures.
- Inappropriate use or bad manufacturing can lead to premature wear of the attachments.
- The functionality of the attachments will be adversely affected by traumas such as grinding and bruxism.
- For the purpose of traceability we advise you to record the lot number of the applied products in the patient file.
- Do not heat items containing titanium.
- Do not use items containing nickel in case of nickel allergy.
- The accessories RE H 79 and H 35 must be used outside the mouth.