

## **REBASING/RELINING A PROSTHESIS WITH PRECI-HORIX**

1. Take a reline impression with the riders in the prosthesis.
2. Remove the riders from the prosthesis and fill up the space of the bar and the riders with plaster and separating liquid.
3. Pour a stone model. Remove the impression material. The use of a reline jig or flask is recommended.
4. Reline in routine technique.
5. Remove the silicone after polymerization.
6. Finish the prosthesis and incorporate new riders.

### **Side effects, warnings and precautions**

- The attachments are intended for single use.
- The products are non-sterile.
- There is a risk of poor fit when patient conditions change.
- Bacterial adhesion can be avoided by applying hygiene measures.
- Inappropriate use or bad manufacturing can lead to premature wear of the attachments.
- The functionality of the attachments will be adversely affected by traumas such as grinding and bruxism.
- For the purpose of traceability we advise you to record the lot number of the applied products in the patient file.
- Do not heat items containing titanium.
- Do not use items containing nickel in case of nickel allergy.
- The accessories RE H 79 and H 35 must be used outside the mouth.