

REPLACING A RIDER IN A PROSTHESIS WITH PRECI-CLIP

1. Cut a length from the **1707** pattern analogue to the length of a rider. Provide the underside of the analogue with retention for fixation in the stone model.
2. Seat the analogue with the space maintainer in the rider in the prosthesis and pour a stone model.
3. Cut the old rider in the prosthesis free and remove it.
4. Place the new **1105/1106/1125/1126** rider with the space maintainer on the model and polymerize it with self-curing acrylic resin.
5. Activate the rider after finishing if necessary (see INFO 107).

Side effects, warnings and precautions

- The attachments are intended for single use.
- The products are non-sterile.
- There is a risk of poor fit when patient conditions change.
- Bacterial adhesion can be avoided by applying hygiene measures.
- Inappropriate use or bad manufacturing can lead to premature wear of the attachments.
- The functionality of the attachments will be adversely affected by traumas such as grinding and bruxism.
- For the purpose of traceability we advise you to record the lot number of the applied products in the patient file.
- Do not heat items containing titanium.
- Do not use items containing nickel in case of nickel allergy.
- The accessories RE H 79 and H 35 must be used outside the mouth.