

REBASING/RELINING A PROSTHESIS WITH PRECI-BALL

1. Take a reline impression with the females in the prosthesis.
2. Seat the **1201 D** model analogue in the females in the prosthesis.
3. Pour a stone model.
4. Complete routine reline procedure using a reline jig or flask.
5. Activate or deactivate the female after finishing and polishing if necessary (see INFO 053).

Side effects, warnings and precautions

- The attachments are intended for single use.
- The products are non-sterile.
- There is a risk of poor fit when patient conditions change.
- Bacterial adhesion can be avoided by applying hygiene measures.
- Inappropriate use or bad manufacturing can lead to premature wear of the attachments.
- The functionality of the attachments will be adversely affected by traumas such as grinding and bruxism.
- For the purpose of traceability we advise you to record the lot number of the applied products in the patient file.
- Do not heat items containing titanium.
- Do not use items containing nickel in case of nickel allergy.
- The accessories RE H 79 and H 35 must be used outside the mouth.