

## **FABRICATION OF A PROSTHESIS WITH PRECI-BALL**

1. Take a full arch impression.
2. Place the **1201 D** model analogue in the impression and pour a stone model.
3. Place the **RA 0055** large space maintainer over the male on the post-coping. Fill up the area between papilla and post-coping with plaster. The space maintainer must cover the edge of the papilla by 1 to 1.5 mm.
4. Fill the female half-way with silicone and seat it over the male. The pvc ring must surround the female segments. Leave the retention zone for acrylic resin on the female free.
5. Place the wax setup on the model. Prepare for investing.
6. Polymerize and finish.
7. Remove the space maintainer after polymerization. Do not remove the pvc ring.
8. Check the retention and activate the female if necessary (see INFO 053).

### **Side effects, warnings and precautions**

- The attachments are intended for single use.
- The products are non-sterile.
- There is a risk of poor fit when patient conditions change.
- Bacterial adhesion can be avoided by applying hygiene measures.
- Inappropriate use or bad manufacturing can lead to premature wear of the attachments.
- The functionality of the attachments will be adversely affected by traumas such as grinding and bruxism.
- For the purpose of traceability we advise you to record the lot number of the applied products in the patient file.
- Do not heat items containing titanium.
- Do not use items containing nickel in case of nickel allergy.
- The accessories RE H 79 and H 35 must be used outside the mouth.