

REPLACING A FEMALE IN A PROSTHESIS WITH PRECI-BALL

1. Press the **1201 D** model analogue into the female to be replaced in the prosthesis.
2. Pour a stone model.
3. Cut the old female out of the prosthesis. Make a lingual opening in the prosthesis.
4. Fill the new **1201 A**, **1205 A** or **1207 A + 694 AKS2** female half-way with silicone and seat it over the model analogue. Use the space maintainer. The pvc ring must surround the female segments. Leave the retention zone for acrylic resin on the female free.
5. Seat the prosthesis on the model and let self-curing acrylic resin flow in.
6. Polymerize and finish.
7. Do not remove the pvc ring.
8. Check retention and activate the female if necessary (see INFO 053).

Side effects, warnings and precautions

- The attachments are intended for single use.
- The products are non-sterile.
- There is a risk of poor fit when patient conditions change.
- Bacterial adhesion can be avoided by applying hygiene measures.
- Inappropriate use or bad manufacturing can lead to premature wear of the attachments.
- The functionality of the attachments will be adversely affected by traumas such as grinding and bruxism.
- For the purpose of traceability we advise you to record the lot number of the applied products in the patient file.
- Do not heat items containing titanium.
- Do not use items containing nickel in case of nickel allergy.
- The accessories RE H 79 and H 35 must be used outside the mouth.