

EXPANDO

Intraoral universal retractor in 3 sizes

1. Check if the cheek shield is cleaned, disinfected, and sterilized.

EXPANDO can only be sterilized at low temperature – with long cycle.
See table below for the chemical reaction of disinfecting agents on EXPANDO.

2. Select the appropriate size for the patient to be treated. The three sizes (1-2-3) are recognized by the 1, 2 or 3 points on the tongue retainer.
3. Stretch the retractor between thumb and forefinger. The tongue retainer must point downwards.
4. Position the first side of the retractor. The bulge on the retractor arm must be on the edge between lip and cheek. Stretch the retractor and position the other side.
5. Check if the patient feels comfortable with the device, and if it does not irritate lips, cheek or tongue.

Chemical reaction of disinfecting agents on EXPANDO

+ RESISTANT		- NON-RESISTANT
+ RESORCINE SOLUTION 1 % PC + MERFEN 2 % + CHLORAMINE 2 %	+ OCTOZONE 1 % PC + LYSOFORM 2 % + ODOL 1.5 %	+ MAKTOL 2 % + LYSOROMINE 2 %
- ZEPHIROL - DIMANINE	- TB-LYSOFORM	- DDT