

FABRICATION OF A NEW PROSTHESIS WITH PRECI-CLIX

CHAIRSIDE PROCEDURE

1. Block out the undercuts of the female with soft wax.
2. Take a full arch impression using a custom-made impression tray.

LABORATORY PROCEDURE

1. Place the **1201 D** model analogue or the **RE H 14 + 1206 C** assembly in the impression.
2. Pour a working model with incorporated model analogues.
3. Make a new secondary construction.

Side effects, warnings and precautions

- The attachments are intended for single use.
- The products are non-sterile.
- There is a risk of poor fit when patient conditions change.
- Bacterial adhesion can be avoided by applying hygiene measures.
- Inappropriate use or bad manufacturing can lead to premature wear of the attachments.
- The functionality of the attachments will be adversely affected by traumas such as grinding and bruxism.
- For the purpose of traceability we advise you to record the lot number of the applied products in the patient file.
- Do not heat items containing titanium.
- Do not use items containing nickel in case of nickel allergy.
- The accessories RE H 79 and H 35 must be used outside the mouth.