

CONSTRUCTION OF REVAX TI EXTRACORONAL M 2 INTO PRECI-CLIX

In case of repeated fracture of spring pins with traditional extracoronar partial dentures, it may be advisable to change the construction into a PRECI-CLIX construction, where the females are the elastic parts.

The female/male keeper of **REVAX RE 0161 TI, RE 0175 TI, RE 0185 TI, RE 0195 TI** and **PRECI-CLIX TI** have identical internal diameters, providing for an easy conversion.

CHAIRSIDE PRODEDURE

1. Remove the female and the **CEKA SITE (# CEKA SITE B)** from the female keeper with a cool cylindrical diamond burr.
2. The cavity must remain cylindrical.
3. Clean and dry the cavity.
4. Cement a **1271 C** titanium male into the cavity with **CEKA SITE**.
5. Remove the excess material after hardening.
6. Remove the male assembly from the prosthesis.
7. Adjust the opening in the prosthesis to the diameter of the new female (4.5 mm diameter and 3.5 mm height), and perforate the acrylic resin denture to the lingual.
8. Press a **1234** female into a **1235** titanium housing with the **1222** insertion tool.
9. Assemble the male with the **1251 B** space maintainer and the female assembly.
10. Mix a small amount of self-cure acrylic resin to a doughy consistency (preferably in another colour than the denture base), place it in the cavity for the female, and seat the prosthesis in the patient's mouth. The patient must not load the prosthesis.
11. Remove the excess material and the space maintainer after hardening of the acrylic resin.

Side effects, warnings and precautions

- The attachments are intended for single use.
- The products are non-sterile.
- There is a risk of poor fit when patient conditions change.
- Bacterial adhesion can be avoided by applying hygiene measures.
- Inappropriate use or bad manufacturing can lead to premature wear of the attachments.
- The functionality of the attachments will be adversely affected by traumas such as grinding and bruxism.
- For the purpose of traceability we advise you to record the lot number of the applied products in the patient file.
- Do not heat items containing titanium.
- Do not use items containing nickel in case of nickel allergy.
- The accessories RE H 79 and H 35 must be used outside the mouth.