

## **REBASING/RELINING A PROSTHESIS WITH PRECI-52**

1. Apply impression material to the tissue side of the prosthesis and bring the prosthesis manually into centric occlusion.
2. Unthread the **1563** (U - Mini  $\varnothing$  3.15 mm) or **1568** (S - Standard  $\varnothing$  3.40 mm) internal male set using the **1570** star-shaped screw-driver.
3. Thread the **1564** polymerization plug into the internal threading of the male housing.
4. Pour a model in stone.
5. Reline as usual using a reline jig or flask.
6. Finish the acrylic of the prosthesis.
7. Unthread the **1564** polymerization plug from the male housing and replace it with an **1563** (U - Mini  $\varnothing$  3.15 mm) or **1568** (S - Standard  $\varnothing$  3.40 mm) internal male set using the **1570** star-shaped screw-driver.
8. Place the prosthesis.
9. If necessary, adjust the retention of the male part by holding the spherical male with an instrument and by tightening or loosening the small retention nut using the **1570** star-shaped screw-driver.

### **Side effects, warnings and precautions**

- The attachments are intended for single use.
- The products are non-sterile.
- There is a risk of poor fit when patient conditions change.
- Bacterial adhesion can be avoided by applying hygiene measures.
- Inappropriate use or bad manufacturing can lead to premature wear of the attachments.
- The functionality of the attachments will be adversely affected by traumas such as grinding and bruxism.
- For the purpose of traceability we advise you to record the lot number of the applied products in the patient file.
- Do not heat items containing titanium.
- Do not use items containing nickel in case of nickel allergy.
- The accessories RE H 79 and H 35 must be used outside the mouth.